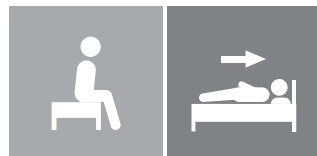
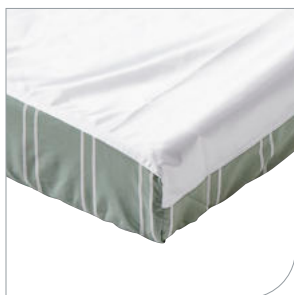
# Instruction for use

IMM1024\_IFU.Version19.2023-12-05

# SatinSheet 2Direction

BaseSheet

**immedia**  
by Etac

# Illustrations

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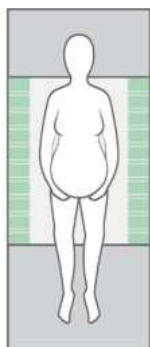
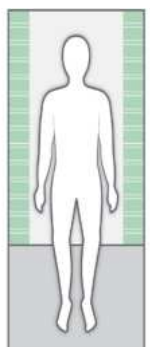
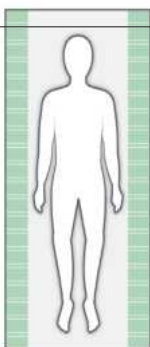
Article no.	BaseSheet - Maxi
IM4181S	SatinSheet 2Direction maxi - small
IM4114S	SatinSheet 2Direction maxi
IM4177S	SatinSheet 2Direction maxi - wide
IM4180S	SatinSheet 2Direction maxi - double
IM4122S	SatinSheet 2Direction 180 cm bed
Article no.	BaseSheet - Maxi Corner
IM4107S	SatinSheet 2Direction Corner
IM4147S	SatinSheet 2Direction Corner - wide
IM4137S	SatinSheet 2Direction Corner - double
IM4197S	SatinSheet 2Direction Corner - double - maxi
Article no.	BaseSheet - Maxi Fit
IM4112S	SatinSheet 2Direction Fit
IM4142S	SatinSheet 2Direction Elastic Fit
<b>BaseSheet - Maxi/Corner/Fit peut être associé à :</b>	
Article no.	4Direction and In2Sheet 4Direction
M4115S	SatinSheet 4Direction - midi
IM4115SH	SatinSheet 4Direction w/handles - midi
IM4140S	In2Sheet 4Direction w/wings - midi
IM4140SH	In2Sheet 4Direction - w/wings + handles - midi
IM4121S	SatinSheet 4Direction - maxi
IM4121SH	SatinSheet 4Direction w/handles - maxi

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N° d'article	Description	SatinSheet 2Direction BaseSheet	SatinSheet DrawSheet and SatinSheet 4Direction
IM4201S	SatinSheet DrawSheet mini set	IM4108S SatinSheet 2Direction mini	IM4116S SatinSheet DrawSheet - mini
IM4210S	SatinSheet In2Sheet mini set	IM4108S SatinSheet 2Direction mini	IM4110S In2Sheet 4Directions w/wings - mini
IM4209S	SatinSheet In2Sheet midi mini set	IM4119S SatinSheet 2Direction midi - small	IM4110S In2Sheet 4Directions w/wings - mini
IM4205S	SatinSheet DrawSheet midi small set	IM4119S SatinSheet 2Direction midi - small	IM4118S SatinSheet DrawSheet - midi
IM4200S	SatinSheet DrawSheet midi set	IM4113S SatinSheet 2Direction midi	IM4118S SatinSheet DrawSheet - midi
IM4207S	SatinSheet 4Direction midi small set	IM4119S SatinSheet 2Direction midi - small	IM4115S SatinSheet 4Direction - midi
IM4204S	SatinSheet 4Direction midi set	IM4113S SatinSheet 2Direction midi	IM4115S SatinSheet 4Direction - midi
IM4202S	SatinSheet DrawSheet maxi midi set	IM4114S SatinSheet 2Direction maxi	IM4118S SatinSheet DrawSheet - midi
IM4203S	SatinSheet 4Direction Fit midi set	IM4112S SatinSheet 2Direction Fit	IM4115S SatinSheet 4Direction - midi
IM4208S	SatinSheet Corner DrawSheet midi set	IM4107S SatinSheet 2Direction Corner	IM4118S SatinSheet DrawSheet - midi
IM4211S	SatinSheet Corner 4Direction midi set	IM4107S SatinSheet 2Direction Corner	IM4115S SatinSheet 4Direction - midi
IM4212S	SatinSheet Corner 4Direction maxi set	IM4107S SatinSheet 2Direction Corner	IM4121S SatinSheet 4Direction - maxi
IM4213S	SatinSheet 4Direction maxi set	IM4114S SatinSheet 2Direction maxi	IM4121S SatinSheet 4Direction - maxi
IM4214S	SatinSheet 4Direction maxi set - wide	IM4177S SatinSheet 2Direction maxi - wide	IM4121S SatinSheet 4Direction - maxi
IM4215S	SatinSheet 4Direction maxi set - double	IM4180S SatinSheet 2Direction maxi - double	IM4121S SatinSheet 4Direction - maxi
IM4216S	SatinSheet 4Direction midi set - wide	IM4176S SatinSheet 2Direction midi - wide	IM4115S SatinSheet 4Direction - midi
IM4217S	SatinSheet 4Direction midi set - double	IM4179S SatinSheet 2Direction midi - double	IM4115S SatinSheet 4Direction - midi
IM4218S	SatinSheet 4Direction maxi midi set	IM4114S SatinSheet 2Direction maxi	IM4115S SatinSheet 4Direction - midi
IM4219S	SatinSheet 4Direction maxi midi set - wide	IM4177S SatinSheet 2Direction maxi - wide	IM4115S SatinSheet 4Direction - midi
IM4220S	SatinSheet 4Direction maxi midi set - double	IM4180S SatinSheet 2Direction maxi - double	IM4115S SatinSheet 4Direction - midi
IM4221S	SatinSheet DrawSheet maxi midi set - wide	IM4177S SatinSheet 2Direction maxi - wide	IM4118S SatinSheet DrawSheet - midi
IM4222S	SatinSheet DrawSheet maxi midi set - double	IM4180S SatinSheet 2Direction maxi - double	IM4118S SatinSheet DrawSheet - midi
IM4223S	SatinSheet DrawSheet midi set - wide	IM4176S SatinSheet 2Direction midi - wide	IM4118S SatinSheet DrawSheet - midi
IM4224S	SatinSheet DrawSheet midi set - double	IM4179S SatinSheet 2Direction midi - double	IM4118S SatinSheet DrawSheet - midi
IM4226S	SatinSheet Corner 4Direction midi set - double	IM4137S SatinSheet 2Direction Corner - double	IM4115S SatinSheet 4Direction - midi
IM4227S	SatinSheet Corner DrawSheet midi set - wide	IM4147S SatinSheet 2Direction Corner - wide	IM4118S SatinSheet DrawSheet - midi
IM4228S	SatinSheet Corner DrawSheet midi set - double	IM4137S SatinSheet 2Direction Corner - double	IM4118S SatinSheet DrawSheet - midi
IM4229S	SatinSheet Corner 4Direction maxi set - wide	IM4147S SatinSheet 2Direction Corner - wide	IM4121S SatinSheet 4Direction - maxi
IM4230S	SatinSheet Corner 4Direction maxi set - double	IM4137S SatinSheet 2Direction Corner - double	IM4121S SatinSheet 4Direction - maxi
IM4231S-RCNL	SatinSheet Corner 4Direction w/handle midi - set with dutch insert	IM4107S SatinSheet 2Direction Corner	IM4115SH SatinSheet 4Direction w/ handles - midi
IM4232S	SatinSheet 4Direction w/handles midi small set	IM4119S SatinSheet 2Direction midi - small	IM4115SH SatinSheet 4Direction w/ handles - midi
IM4233S	SatinSheet 4Direction midi set - wide	IM4176S SatinSheet 2Direction midi - wide	IM4115SH SatinSheet 4Direction w/ handles - midi
IM4234S	SatinSheet 4Direction w/handles maxi set - wide	IM4177S SatinSheet 2Direction maxi - wide	IM4121SH SatinSheet 4Direction w/ handles - maxi
IM4235S	SatinSheet 4Direction w/handles midi set - double	IM4179S SatinSheet 2Direction midi - double	IM4115SH SatinSheet 4Direction w/ handles - midi
IM4236S	SatinSheet 4Direction w/handles maxi set - double	IM4180S SatinSheet 2Direction maxi - double	IM4121SH SatinSheet 4Direction w/ handles - maxi
IM4238S	SatinSheet 4Direction maxi set - small	IM4181S SatinSheet 2Direction maxi - small	IM4121S SatinSheet 4Direction - maxi
IM4239S	SatinSheet 4Direction w/handles maxi set - small	IM4181S SatinSheet 2Direction maxi - small	IM4121SH SatinSheet 4Direction w/ handles - maxi
IM4242S	SatinSheet DrawSheet mini set - wide	IM4175S SatinSheet 2Direction mini - wide	IM4116S SatinSheet DrawSheet - mini
IM4243S	SatinSheet DrawSheet mini set - double	IM4178S SatinSheet 2Direction mini - double	IM4116S SatinSheet DrawSheet - mini
IM4249S	SatinSheet In2Sheet midi midi set	IM4119S SatinSheet 2Direction midi - small	IM4140S In2Sheet 4Direction w. wings IM4245S SatinSheet Corner DrawSheet midi set IM4208S SatinSheet Corner DrawSheet midi set

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