THUASNE

# MalleoSoft

<b>fr</b>	Chevillère élastique proprioceptive .....	7
<b>en</b>	Proprioceptive elastic ankle support .....	8
<b>de</b>	Elastische propriozeptive Knöchelbandage .....	9
<b>nl</b>	Proprioceptieve elastische enkelbrace .....	11
<b>it</b>	Cavigliera elastica propriocectiva .....	12
<b>es</b>	Tobillera elástica propioceptiva .....	14
<b>pt</b>	Tornozeleira elástica propriocectiva .....	15
<b>da</b>	Proprioceptisk, elastisk ankelbind .....	17
<b>fi</b>	Proprioseptiivinen joustava nilkkatuki .....	18
<b>sv</b>	Proprioceptiv elastisk ankelortos .....	20
<b>el</b>	Ελαστική ιδιοδεκτική επιστραγαλίδα .....	21
<b>cs</b>	Elastická proprioceptivní kotníková ortéza .....	22
<b>pl</b>	Elastyczna proprioceptywna orteza skokowo-stopowa .....	24
<b>lv</b>	Proprioceptīvs elastīgs potītes pārsējs .....	25
<b>lt</b>	Tamprus propriocepcija gerinantis čiurnos įtvaras .....	27
<b>et</b>	Propriotseptiivne elastne hüppeliigesetugi .....	28
<b>sl</b>	Elastična proprioceptivna opornica .....	30
<b>sk</b>	Elastická proprioceptívna členková ortéza .....	31
<b>hu</b>	Proprioceptív rugalmas bokarögzítő .....	33
<b>bg</b>	Проприоцептивна еластична шина за глезен .....	34
<b>ro</b>	Glezniere elastică proprioceptivă .....	35
<b>ru</b>	Эластичный проприоцептивный ортез на лодыжку .....	37
<b>hr</b>	Elastični proprioceptivni steznik za gležanj .....	38
<b>zh</b>	彈力本体感护踝 .....	40
<b>ar</b>	دعامة كاحل مرنة متعلقة باستقبال الحس العميق .....	43

	1	19 - 21 cm
	2	21 - 23 cm
	3	23 - 25 cm
	4	25 - 27 cm
	5	27 - 30 cm

<b>fr</b>	Tour de cheville
<b>en</b>	Ankle circumference
<b>de</b>	Knöchelumfang
<b>nl</b>	Omvang van de enkel
<b>it</b>	Giro caviglia
<b>es</b>	Circunferencia del tobillo
<b>pt</b>	Perímetro do tornozelo
<b>da</b>	Anklens omkreds
<b>fi</b>	Nilkanympärys
<b>sv</b>	Runt fotleden
<b>el</b>	Περίμετρος αστραγάλου
<b>cs</b>	Obvod kotníku
<b>pl</b>	Obwód kostki
<b>lv</b>	Potītes apkārtmērs
<b>lt</b>	Kulkšnies apimtis
<b>et</b>	Pahkluu ümbermõõt
<b>sl</b>	Obseg gležnja
<b>sk</b>	Obvod členka
<b>hu</b>	Boka kerülete
<b>bg</b>	Обиколка на глезена
<b>ro</b>	Circumferința gleznei
<b>ru</b>	Окружность голени
<b>hr</b>	Opseg gležnja
<b>zh</b>	脚踝尺寸
<b>ar</b>	محيط الكاحل

<b>fr</b>	Maintien
<b>en</b>	Support
<b>de</b>	Halt
<b>nl</b>	Fixatie
<b>it</b>	Sostegno
<b>es</b>	Sujeción
<b>pt</b>	Manutenção
<b>da</b>	Støtte
<b>fi</b>	Tuki
<b>sv</b>	Stöd
<b>el</b>	Συγκράτηση
<b>cs</b>	Stabilizace
<b>pl</b>	Podtrzymywanie
<b>lv</b>	Atbalsts
<b>lt</b>	Palaikymas
<b>et</b>	Hooldus
<b>sl</b>	Oprijem
<b>sk</b>	Stabilizácia
<b>hu</b>	Támasz
<b>bg</b>	Поддръжка
<b>ro</b>	Suținere
<b>ru</b>	Стабилизация
<b>hr</b>	Potpora
<b>zh</b>	穩定
<b>ar</b>	الدعم

Compression	Effet proprioceptif
Compression	Proprioceptive effect
Kompression	Proprioceptive Wirkung
Compressie	Proprioceptieve werking
Compressione	Effetto propriocettivo
Compresión	Efecto propioceptivo
Compressão	Efeito proprioceptivo
Kompression	Proprioceptisk effekt
Kompressio	Liikeaistiin kohdistuva vaikutus
Kompression	Proprioceptiv effekt
Συμπίεση	Αποτέλεσμα ιδιοδεκτικότητας
Komprese	Proprioceptivní účinek
Kompresja	Propriocepja
Kompresija	Proprioceptivus efekts
Kompresija	Propriocepçinis poveikis
Kompressioon	Proprioceptiivne toime
Kompresija	Proprioceptivni učinek
Kompresia	Proprioceptivny učinok
Kompresszió	Proprioceptívus hatás
Компрессия	Проприоцептивный эффект
Compresie	Efect proprioceptiv
Компрессия	Проприоцептивный эффект
Kompresija	Učinak proprioceptivnog vježbanja
压紧	本体感

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---

**en**

## **PROPRIOCEPTIVE ELASTIC ANKLE SUPPORT**

#### **Description/Destination**

This device is a proprioceptive elastic ankle brace providing joint support.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

#### **Composition**

Polyamide - Cotton - Elastane.

#### **Properties/Mode of action**

Elastic compressive knitting (class 2 - 15-20 mmHg / 20,1-27 hPa).

Comfort zone over the ankle bones.

#### **Indications**

Post-operative, post-traumatic residual pain and/or edema.

Proprioceptive support for rehabilitation.

Sport activities resumption.

Compression for scar/burns healing.

#### **Contraindications**

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin or an open wound without an adequate dressing.

Do not use in the event of known allergy to any of the components.

History of venous or lymphatic disorders.

#### **Precautions**

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

If the support is worn for prolonged periods of time, ensure that blood circulation is not impeded.

#### **Undesirable side-effect**

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Possible risk of venous thrombosis.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

#### **Instructions for use/Application**

Pull the ankle brace over the ankle joint, like a sock.

Place the malleoli correctly, on the comfort zones.

#### **Care/Maintenance**

Product can be washed in accordance with the conditions shown on this notice and on the label.

Machine washable at 40°C (delicate programme). Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

#### **Storage**

Store at room temperature, preferably in the original packaging.

#### **Disposal**

Dispose of in accordance with local regulations.

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