DONJOY®

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE
The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to:

- Read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS

- Reducible unilateral inguinal hernias
- Pre-operative use

CONTRAINDICATIONS

- None

WARNING & PRECAUTIONS

- We recommend that the first application be made under supervision of a health care professional.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- If worn directly on the skin, the skin should be free from injuries, moisture, oils, creams, gels or similar residues.

APPLICATION INSTRUCTIONS

- 1) Apply foam pad to right/left of rigid section according to location of inguinal hernia.
- 2) Put belt on hips and fasten waist strap in front
- 3) Tighten belt by pulling on grey velvet. Insert velvet into loop to keep tightening.
- 4) Rotate belt to position foam pad on the hernia
- 5a) Insert the under-thigh strap into white buckle on the side of the inguinal hernia. Pass strap under the thigh and attach to the central section.
- 5b) Adjust the fit using the white velcro tab.

CARE
Wash in lukewarm water with a mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry-clean. Do not iron.

COMPOSITION
, Cotton, Nylon, Polyester, Polyurethane, Polyamide, Ethylene Vinyl Acetate (EVA), Rubber, Plastic (Acetal & PA)

LATEX CAUTION: THIS PRODUCT CONTAINS NATURAL RUBBER LATEX WHICH MAY CAUSE ALLERGIC REACTIONS.

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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