# COLLIERS CERVICAUX C1 & C2 CERVICAL COLLARS C1 & C2

6625/6626/6627/6628/6629/6630/6631/6632/6642  
/6643/6644/6645

6633/6634/6635/6636/6637/6638/6639/6640/6641

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EN | Instructions for Use

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SV | Bruksanvisning

NO | Bruksanvisning

DA | Bruksanvisning

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## FR COLLIER CERVICAL SOUPLE C1

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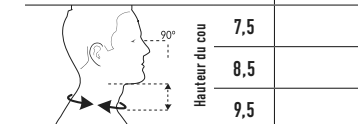
## EN C1 SOFT CERVICAL COLLAR

If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopedic consultant for advice.

### INDICATIONS

For minor neck injuries or conditions that require motion restriction of the cervical spine such as:

- Neck extension injuries (whiplash).
- Cervical spondylosis / Cervical Osteoarthritis (OA).
- Rheumatoid arthritis (RA) of cervical spine.
- Herniated cervical disc.

### PERFORMANCE

The device restricts range of motion of the cervical spine.

### CONTRAINDICATION

The orthosis must not be used for traumatic injuries, apart from mild strain.

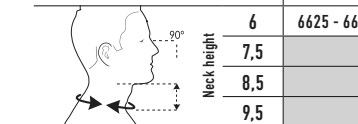
### WARNING • PRECAUTION OF USE

The orthosis must not be used in direct contact with a damaged skin. Do not wear for extended periods without medical follow-up. The orthosis is a single patient use device. Do not use for multiple patients. If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, contact your healthcare professional. The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which

### MEASUREMENT AND SIZE CHART

Height of the collar: measure the height between the sternal fork and under the chin (in cm), with the patient looking horizontally. Necklace size: measure the circumference at the base of the neck (in cm)

Size	Neck circumference				
	0	1	2	3	4
6	6625 - 6629 - 6642				
7,5			6626 - 6630 - 6643		
8,5			6627 - 6631 - 6644		
9,5			6628 - 6632 - 6645		

## EN C2 ADJUSTABLE CERVICAL COLLAR

If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopedic consultant for advice.

### INDICATIONS

For minor neck injuries or conditions that require motion restriction of the cervical spine such as:

- Neck extension injuries (whiplash).
- Cervical spondylosis / Cervical Osteoarthritis (OA).
- Rheumatoid arthritis (RA) of cervical spine.
- Herniated cervical disc.

### PERFORMANCE

The device restricts range of motion of the cervical spine.

### CONTRAINDICATION

The orthosis must not be used for traumatic injuries, apart from mild strain.

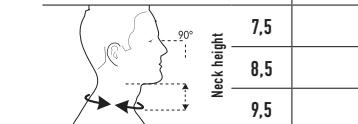
### WARNING • PRECAUTION OF USE

The orthosis must not be used in direct contact with a damaged skin. Do not wear for extended periods without medical follow-up. The orthosis is a single patient use device. Do not use for multiple patients. If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, contact your healthcare professional. The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which

### MEASUREMENT AND SIZE CHART

Height of the collar: measure the height between the sternal fork and under the chin (in cm), with the patient looking horizontally. Necklace size: measure the circumference at the base of the neck (in cm)

Size	Neck circumference			
	1	2	3	4
7,5	6633 - 6636 - 6639			
8,5	6634 - 6637 - 6640			
9,5	6635 - 6638 - 6641			

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