
# SET DE POSE DIU EVOLUTION<sup>®</sup>

## IUD KIT EVOLUTION<sup>®</sup>

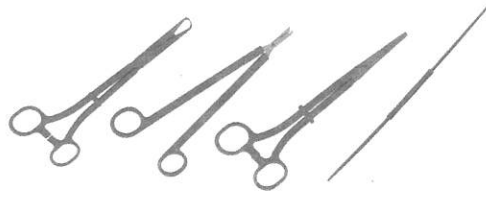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### DESCRIPTION

The IUD kit EVOLUTION is composed :

- Cut-thread scissor with notch – length 22cm in stainless steel
- Long Evolution forceps – length 25cm in Polyamid and fiberglass
- Pozzi forceps – length 25 cm in Polyamid and fiberglass
- Double hystrometer – length 30 cm in Polyethylene
- The hystrometer has two flexible ends, including tips with progressive dilations for all type of IUD insertion
- One end with progressive dilation size CH8 (diameter 2,66mm) to CH12 (diameter 4 mm)
- Other end with progressive dilation size CH10 (diameter 3,3mm) to CH14 (diameter 4,7mm).

Each end is protected by a polyethylene tube to ensure its asepticity.

### INSTRUCTIONS FOR USE

After inserting a speculum, use the long forceps Evolution to disinfect the cervix.  
Grasp the cervix with the Pozzi cervical tenaculum to straighten the uterus and position it with the axis of the vagina. Use the uterine sound by choosing the part indicated according to the IUD model.  
Gently introduce the uterine sound into the cervical canal, to the bottom of the uterus to assess its depth and check the direction of the uterus, remove it gently. Insert the IUD according to manufacturer instructions.  
Cut the string of the IUD using the provided long scissors.

### PRODUCT DISPOSAL AFTER USE

After the procedure, dispose of any used product and its packaging in strict compliance with your facility's procedures for treating medical waste.

### RECOMMENDATIONS

Read IUD manufacturer instructions before use  
This medical device is intended for use only by health care professionals  
**DO NOT USE** if the package has been opened or damaged or if there is a defect caused by poor shipping, storage, or handling that could affect its use.

**NOTICE : THESE DEVICES ARE STRICTLY FOR SINGLE USE ONLY AND MAY NOT BE REUSED; REUSE COULD ALTER PERFORMANCE OR IMPOSE A SERIOUS RISK OF CONTAMINATION.**

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