**Lansinoh.**

**EN**

### POST BIRTH WASH BOTTLE

For gentle washing of the perineal area to soothe post-birth discomfort.

**How to use:**

- Fill bottle with warm water. Screw cap on securely and extend nozzle firmly.

- While on the toilet, invert bottle between legs aim the tip towards the perineal area.

- Squeeze bottle gently and repeat as needed.

**Cleaning instructions**

- After each use, disassemble and clean with mild soap and warm water only (do not boil).

- Leave to dry before storing.

Distributed by: Lansinoh Laboratories UK Ltd, Ground Floor, West Mayesbrook House, Lawnswood Business Park, Leeds, LS16 6QY, UK | lansinoh.co.uk

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Distribuit de: Lansinoh Laboratories UK Ltd, Ground Floor, West Mayesbrook House, Lawnswood Business Park, Leeds, LS16 6QY, UK | www.lansinoh.ro

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Jakelija: Lansinoh Laboratories UK Ltd, Ground Floor, West Mayesbrook House, Lawnswood Business Park, Leeds, LS16 6QY, UK | www.lansinoh.fi

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Distribueret af: Lansinoh Laboratories UK Ltd, Ground Floor, West Mayesbrook House, Lawnswood Business Park, Leeds, LS16 6QY, UK | www.lansinohdanmark.com

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Distribuerad av: Lansinoh Laboratories UK Ltd, Ground Floor, West Mayesbrook House, Lawnswood Business Park, Leeds, LS16 6QY, UK | www.lansinohdanmark.com