PODERM

PROFESSIONAL

(FR) NOTRE HISTOIRE

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(EN) OUR HISTORY

The PODERM® laboratory specializes in medical innovations for the health of feet and nails, and offers a wide range of serums and protective braces. The products are developed in Switzerland, in a laboratory that combines professional skill and science. poderm.com

BREATHABLE FABRIC: the PODERM® fabric has been designed especially to allow skin to breathe, sweat to escape and the natural ingredients in the PODERM® Gel to pass through.

PODERM® GEL: Immediately soothes pain, limits friction, spreads pressure and absorbs micro-impacts for optimum comfort. It prevents calluses and promotes their disappearance.

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(EN) Directions for use: Place the product directly against the skin. Wash the product by hand with warm water and allow to dry. Do not use for more than 16 hours per day. Do not apply to an open wound. Diabetics and people suffering from blood circulation problems must seek advice from a healthcare professional before use.

Medical device.

Date of obtention of CE mark: September 2021.

PODERM® HARD/SOFT CORN PROTECTION. Washable and reusable medical device with gel and Polyamide/Lycra fabric.

What is a hard/soft - corn? The corn (hard or soft) is a frequent pathology of the forefoot caused by repeated friction and revealed by pain or the presence of (plantar) calluses/corns.

Why choose PODERM® HARD/SOFT CORN PROTECTION? PODERM® HARD/SOFT CORN PROTECTION forms a protection around the toe and reduces friction and pressure. The internal medical gel cushion provides immediate relief and helps to prevent and soften calluses. Its reduced size adapts to all styles of flexible footwear.

What are the symptoms of a hard/soft corn? Pain present in a toe. Appearance of redness and formation of calluses. Appearance of a bony protuberance on the upper part of the toes.

When should you use PODERM® HARD/SOFT CORN PROTECTION? The symptoms are recent. There is a need to protect the toes from pressure and friction.

INSTRUCTIONS FOR USE

Identify the areas of the toes most exposed to pressure, friction, or skin dryness that could lead to lesions.

Select the most suitable diameter and fabric type according to the area to be protected. Clean and thoroughly dry the application area.

Remove the product from its original packaging. Keep the packaging to store the product when not in use.

Measure and cut the desired length of the tube.

Open the tube by slightly widening its diameter so that it comfortably fits the toe. The material is elastic, but do not stretch it beyond twice its original size.

Both ends of the tube can be applied directly to the skin.

The product can be fully covered with socks, stockings, or shoes without causing discomfort.

WARNINGS AND PRECAUTIONS

Do not apply directly to skin with open or oozing wounds.

For external use only. Do not use for more than 16 hours per day. Regularly inspect the condition of the skin.

Discontinue use immediately and consult a healthcare professional in the event of:

- Loss of skin integrity (lesions, deterioration).
- Suspected or confirmed deep tissue injury.
- Skin irritation.
- Appearance of redness or heat-related rashes.

• Ensure the skin is completely dry before each application or reapplication.

• People with diabetes or circulatory disorders should seek medical advice before use.

PODERM® DAMAGED NAIL BY FUNGUS AND TRAUMATISM PROTECTION. Washable and reusable medical device.

What is a damaged nail? A damaged nail is a common foot problem that can be caused by repeated friction in enclosed shoes or a deformation of the toes (claw toes, hallux extensus, etc.). This problem is revealed by a change in the visual appearance of the nail, thickening or a detachment of the nail plate.

Why choose the PODERM® DAMAGED NAIL BY FUNGUS AND TRAUMATISM PROTECTION? PODERM® DAMAGED NAIL PROTECTION forms a protection around the nail and prevents the nail deteriorating and/or detaching. Its reduced size adapts to all styles of flexible footwear.

What are the symptoms of a damaged nail? Change in the visual appearance of the nail, detachment of the nail plate.

When should I choose the PODERM® DAMAGED NAIL BY FUNGUS AND TRAUMATISM PROTECTION? The symptoms are recent. You want to protect your nail and encourage healthy regrowth.

INSTRUCTIONS FOR USE

Identify the areas of the toes most exposed to pressure, friction, or skin dryness that could lead to lesions.

Select the most suitable diameter and fabric type according to the area to be protected. Clean and thoroughly dry the application area.

Remove the product from its original packaging. Keep the packaging to store the product when not in use.

Measure and cut the desired length of the tube.

Open the tube by slightly widening its diameter so that it comfortably fits the toe. The material is elastic, but do not stretch it beyond twice its original size.

Both ends of the tube can be applied directly to the skin.

The product can be fully covered with socks, stockings, or shoes without causing discomfort.

WARNINGS AND PRECAUTIONS

Do not apply directly to skin with open or oozing wounds.

For external use only. Do not use for more than 16 hours per day. Regularly inspect the condition of the skin.

Discontinue use immediately and consult a healthcare professional in the event of:

- Loss of skin integrity (lesions, deterioration).
- Suspected or confirmed deep tissue injury.
- Skin irritation.
- Appearance of redness or heat-related rashes.

• Ensure the skin is completely dry before each application or reapplication.

• People with diabetes or circulatory disorders should seek medical advice before use.

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