# Bandelettes nasales - Neusstrips - Nasal stripes - Nasenstreifen

## FR Sommeil - ANTI-RONFLEMENT

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## GB Sleep - ANTI-SNORING

**Causes of snoring**  
The nose is one of the major culprits when it comes to snoring:

- either due to its morphology (narrow nose with pinched in structure, reducing the diameter of the nostrils),
  - or due to pathological reasons (inflammation: rhinitis, infectious or allergic or structural rhinosinusitis: nasal septum deviation, large concha).
- In both cases, widening the nasal passage improves the air flow, providing greater respiratory comfort.  
By encouraging nose breathing and preventing noisier mouth breathing, Quies® nasal strips provide an immediate feeling of easier breathing and reduce snoring.

Transparent, flexible and very comfortable. Also suitable for sports activities. Perfect adhesion to the wings of the nose for many hours thanks to a semi-rigid PET layer and an acrylic adhesive.

**DIRECTIONS FOR USE**  
Clean and dry your nose thoroughly. Remove the protective film. Apply the strip on the central part of the nose by pressing with your fingers so that it adheres well to the skin.  
To remove the strip, moisten it with water and gently remove it by pulling on one side.  
For sports: apply 10 minutes before starting the activity to improve the seal in case of perspiration.

**COMPOSITION**  
PET plastic carrier coated with a biocompatible adhesive.

- WARNINGS**
- External use
  - Single use

- Apply on healthy, clean and dry, non-irritated or sunburnt skin
- Do not use for more than 12 hours a day
- After use, clean the contact areas with soapy water
- If used continuously, interrupt for a few days on a regular basis
- Do not use in children under 3 years of age
- Keep out of reach of children

**SIDE EFFECTS**  
In case of hypersensitivity, the product may cause irritation. Discontinue use in case of obvious irritation or hypersensitivity to the product.

**STORAGE METHOD**  
Store the nasal strips inside the box. Store at room temperature, in a clean and dry place. Do not keep past the expiration date.

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