THUASNE

## Ortel C1 Classic

fr	Collier cervical souple.....	6
en	Soft cervical collar.....	7
de	Weiche Halskrause.....	8
nl	Flexibele nekbrace.....	9
it	Collare cervicale morbido.....	11
es	Collarín cervical flexible.....	12
pt	Colar cervical flexível.....	13
da	Smidig nakkekrave.....	14
fi	Taipuisa kaulatuki.....	16
sv	Mjuk nackkrage.....	17
el	Αυχενικό κολάρο μαλακό.....	18
cs	Pružný krční límec.....	19
pl	Kołnierz elastyczny do unieruchomienia szyi.....	21
lv	Mīksta kakla ortoze.....	22
lt	Lankstus kaklo įtvaras.....	23
et	Pehme emakakaela krae.....	24
sl	Opornica za vrat, prožna.....	26
sk	Ohybný krčný golier.....	27
hu	Lágy nyakmerevítő.....	28
bg	Мекa цервикална яка.....	29
ro	Guler cervical flexibil.....	31
ru	Мягкий шейный воротник.....	32
hr	Mekana ortoza za vratnu kralježnicu.....	33
zh	软质颈圈.....	35
ar	طوق للعتق مرتن.....	36



	a	b		
0	24 - 29 cm			
1	28 - 33 cm	6 cm	9 cm	11 cm
2	34 - 39 cm			
3	40 - 46 cm	-		

<b>fr</b>	(a) Tour de cou (b) Hauteur de cou	<b>lv</b>	(a) Kakla apkārtmērs (b) Kakla augstums
<b>en</b>	(a) Neck circumference (b) Neck height	<b>lt</b>	(a) Kaklo apimtis (a) Kaklo aukštis
<b>de</b>	(a) Halsumfang (b) Halshöhe	<b>et</b>	(a) Kaela ümbermõõt (b) Kaela pikkus
<b>nl</b>	(a) Omtrek van de nek (b) Hoogte van de nek	<b>sl</b>	(a) Obseg vratu (b) Dolžina vratu
<b>it</b>	(a) Circonferenza del collo (b) Altezza del collo	<b>sk</b>	(a) Obvod krku (b) Výška krku
<b>es</b>	(a) Circunferencia del cuello (b) Altura del cuello	<b>hu</b>	(a) Nyak kerülete (b) Nyak magassága
<b>pt</b>	(a) Perímetro do pescoço (b) Altura do pescoço	<b>bg</b>	(a) Размер на яката (b) Височина на яката
<b>da</b>	(a) Halsens omkreds (b) Halsens højde	<b>ro</b>	(a) Circumferința gâtului (b) Înălțimea gâtului
<b>fi</b>	(a) Kaulan ympäryys (b) Kaulan korkeus	<b>ru</b>	(a) Обхват шеи (b) Высота шеи
<b>sv</b>	(a) Runt halsen (b) I höjd med halsen	<b>hr</b>	(a) Opseg vrata (b) Visina vrata
<b>el</b>	(a) Περίμετρος λαιμού (b) Ύψος λαιμού	<b>zh</b>	( a ) 颈围 ( b ) 颈高
<b>cs</b>	(a) Obvod krku (b) Výška krku	<b>ar</b>	محيط العنق (a) ارتفاع العنق (b)
<b>pl</b>	(a) Obwód szyi (a) Wysokość szyi		

<b>fr</b>	Maintien	Effet proprioceptif
<b>en</b>	Support	Proprioceptive effect
<b>de</b>	Halt	Propriozeptive Wirkung
<b>nl</b>	Fixatie	Proprioceptieve werking
<b>it</b>	Sostegno	Effetto proprioceetivo
<b>es</b>	Sujeción	Efecto propioceptivo
<b>pt</b>	Manutenção	Efeito proprioceptivo
<b>da</b>	Støtte	Proprioceptisk effekt
<b>fi</b>	Tuki	Liikeaistiin kohdistuva vaikutus
<b>sv</b>	Stöd	Proprioceptiv effekt
<b>el</b>	Συγκράτηση	Αποτελεσμα ιδιοδεκτικότητας
<b>cs</b>	Stabilizace	Proprioceptivní účinek
<b>pl</b>	Podtrzymywanie	Propriocepcja
<b>lv</b>	Atbalsts	Proprioceptīvs efekts
<b>lt</b>	Palaikymas	Proprioceptinis poveikis
<b>et</b>	Hooldus	Proprioceptiivne toime
<b>sl</b>	Oprajem	Proprioceptivni učinek
<b>sk</b>	Podpora	Proprioceptivny účinok
<b>hu</b>	Támasz	Proprioceptív hatás
<b>bg</b>	Поддръжка	Проприоцептивен ефект
<b>ro</b>	Susținere	Efect proprioceptiv
<b>ru</b>	Стабилизация	Проприоцептивный эффект
<b>hr</b>	Potpورا	Učinak proprioceptivnog vježbanja
<b>zh</b>	稳定	本体感
<b>ar</b>	الدعم	مفعول استقبال الحس العميق

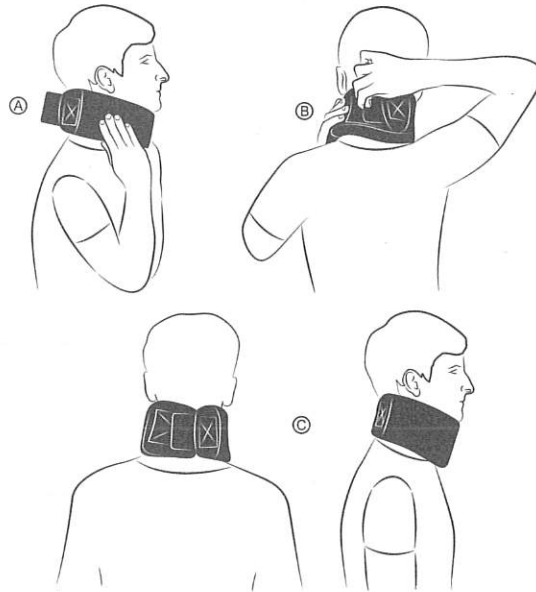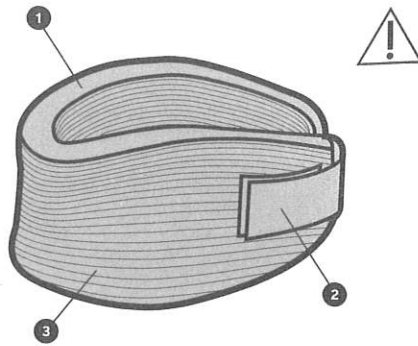4

**MD**

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## SOFT CERVICAL COLLAR

### Description/Destination

Rounded edges for appropriate comfort. ①

Precise self-fastening adjustment. ②

Convenient removable cover. ③

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

### Composition

Cotton - polyamide - polyester - polyurethane.

### Properties/Mode of action

Support.

Proprioceptive effect.

### Indications

Cervical arthritis.

Cervical syndrome.

Dropped head syndrome.

Mild whiplash/mild sprain.

Post-operative use.

Rehabilitation after wearing a rigid cervical collar.

Torticollis (stiff neck).

### Contraindications

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Do not use in the event of traumatic injuries: severe sprain, fracture.

Do not use for prolonged wear without medical supervision.

Do not use the product if the diagnosis has not been confirmed.

### Precautions

Strictly comply with your healthcare professional's prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, abnormal sensations, remove the device and consult a healthcare professional.

Choose the appropriate size to fit the patient, referring to the size chart.

Verify the product's integrity before every use.

It is recommended that a healthcare professional supervises the first application.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device if it is damaged.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

7